SUNLED

I

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Bon travail !

GB

Thank you for choosing STERN PODIA products.

Before using this unit, it is good practice to read this handbook carefully and keep it safely.

Doing so will provide you with a full understanding of the unit, allow you to make the best use of its potentiality and maintain top-level performance, safety, and reliability throughout its working life.

And, whatever circumstances may arise, you can count on the professionalism of your S.M.D. dealer, the expertise of its technicians and our reliability as constructors.

We wish you all the best for your work!

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GB TABLE OF CONTENTS

TECHNICAL SPECIFICATIONS	Page 6
DIMENSIONS	Page 6
WARNINGS	Page 9
SYMBOLS AND TECHNICAL DATA	Page 10
INSTALLATION	Page 11
DESCRIPTION OF PARTS-INSTRUCTIONS FOR USE	Page 12
EVERYDAY MAINTENANCE	Page 13
CLEANING AND DISINFECTION	Page 13
TROUBLESHOOTING	Page 13
CONFORMITY DECLARATION	Page 14

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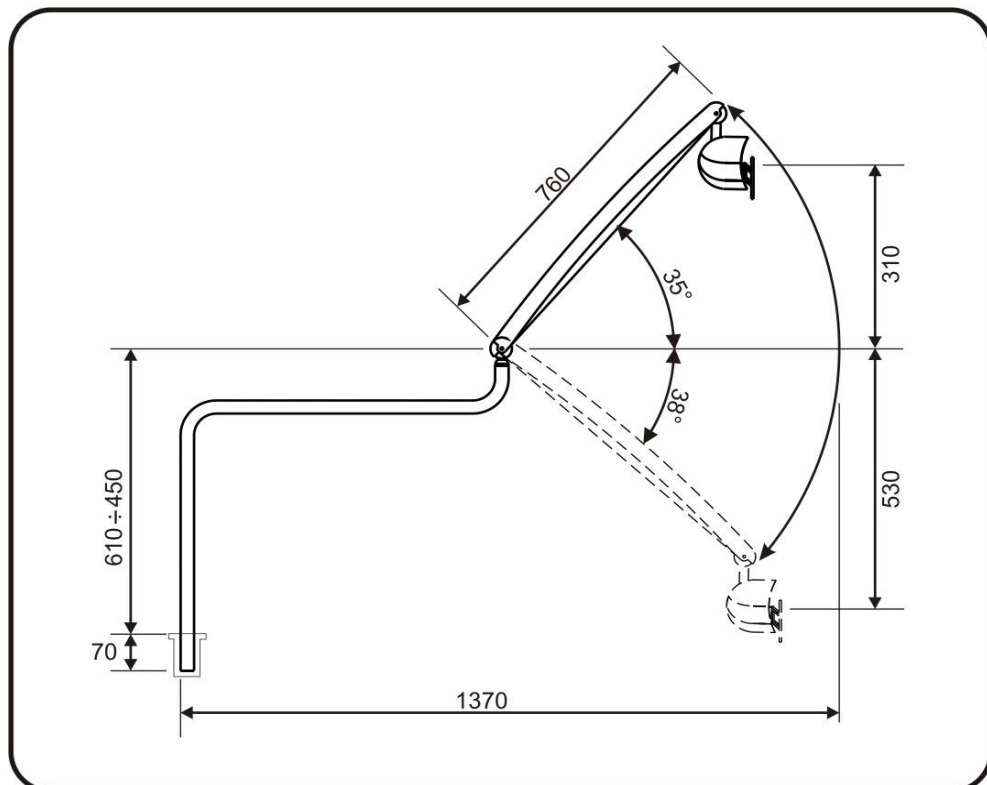
SUNLED TECHNICAL SPECIFICATIONS

Power supply	24 VAC
Light source	7 led x 3W (used 70%)
Average duration	30.000/50.000 hours
Max. power	22VA
Light flow	25.000@ 1 m.
Colour temperature	6.300 °K
CRI (colour resolution index)	91
Total lamp weight	6 kg.
Operating temperature range	10-35°
Operating humidity range	30-90%
Storage and transport temperature	-10°÷40°
Storage and transport humidity	30-90%

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CB WARNINGS

Purpose of use:

Device used for lightening the treatment area during podiatry interventions. Its light emission features allow accurate identification of small details, without raising the temperature of the lit area. The SUN LED lamp may be installed on a specific control and supply device, such as chiropractic units, or with an electrical system that complies with IEC 364-1 norms and the national installation regulations for electrical systems in premises adapted for medical use (check that the supply voltage, shown on the information plate, corresponds to that of the mains supply). Use the lamp in suitable premises only, equipped and constructed in compliance with the regulations in force regarding safety (IEC 60364-7-710). Do not remove the labels attached to the device. Should these deteriorate, ensure that they are replaced. Compliance with directive 93/42/CEE "Medical devices Directive", with the directive 2006/95/CEE "Low Tension Directive" and to the directive 2004/108/CEE "Electromagnetic Compatibility Directive" and subsequent amendments. Do not carry out any maintenance operations on the lamp whilst it is connected to the power supply. Disconnect the power supply cable from the socket before opening the device. The insufficient performance of the lamp does not in any way compromise the safety of the patient. Do not attempt any maintenance operations or replacement of any parts different from those shown in the handbook. Do not direct the light beam directly or indirectly into anyone's eyes, the high intensity of the light could damage the eyesight.

Climate:

Under extreme conditions (heat, cold, humidity) it is advisable to let a few hours pass between unpacking the equipment and using it for the first time. This precaution provides the time needed to eliminate any condensation that may have formed inside the packaging.

S.M.D. cannot be held liable for any damages deriving from failure to observe the warnings and instructions contained in this handbook.

WARNING: any modification is not allowed

WARNING: not modify the device without the manufacturer's permission

WARNING: in case of modification of the device, you must perform examinations and tests in order to ensure its continued safe use

Upon request SMD Srl will provide circuit diagrams, the parts list, descriptions, calibration instructions or other information to assist the After Sales Service in repairing parts of the device indicated by SMD Srl as serviceable by the After Sales Service. In relation to photobiological safety, SUN LED is classified with risk absent.

WARNING: The device cannot be used in environments with explosive atmospheres or fire risk.

Warnings for photo biological risk

According with the EN 62471, the device is classified as risk level 1 for the Blue radiation. In particular, the device has the following emission data at a distance of 200mm. The device is classified risk free to UV, IR and retina.

Danger	Emission	Classification
Actinic UV	0.000000W·m ²	Exempt
Near UV	0.0000W·m ²	Exempt
Blue Light	4921,0 W·m ⁻² ·sr ⁻¹	Group 1
Blue Light small source	43,73 W·m ⁻²	Group 1
Retinal Thermal	4764 W·m ⁻² ·sr ⁻¹	Exempt
IR Radiation	3,694 W·m ⁻² ·sr ⁻¹	Exempt

Photo biological risk: warnings for a correct use

The lamp must not be directly watched.

During the maintenance, disconnect the lamp from the power supply.

If you are to perform tasks with eyes looking directly at the source, you must use the appropriate PPE such as goggles with filter systems and radiation shielding for the Blue radiation.

It is forbidden to change the brightness levels and alter the LED source.

The lamp must be handled with switch in the off position and turned on only upon placement. Moves the lamp is only possible if patients are provided with the appropriate eye protection to safeguard their health.

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Are devices of Class I, classification rule 13, annex VIII of the 745/2017 MDR (Medical Device Regulation).
They comply with the essential requirements of the 745/2017 MDR (Medical Device Regulation).
They comply with the RoHS 2011/65/EU Directive (restriction of the use of hazardous substances in electrical and electronic equipment)
They comply with the RED 2014/53/EU Directive (Radio Equipment Directive)

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