**D**

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¡Buen trabajo!

GB

Thank you for choosing STERN PODIA products.

The equipment you have purchased has been designed and manufactured to meet your present needs and, thanks to its in-built versatility and adaptability, your future needs too.

Before using this unit, it is good practice to read this handbook carefully and keep it safely.

Doing so will provide you with a full understanding of the unit, allow you to make the best use of its potentiality and maintain top-level performance, safety, and reliability throughout its working life.

And, whatever circumstances may arise, you can count on the professionalism of your S.M.D. dealer, the expertise of its technicians and our reliability as constructors.

We wish you all the best for your work!

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GB TABLE OF CONTENTS

TECHNICAL SPECIFICATIONS	Page 7
KEY TO SYMBOLS	Page 8
GENERAL DESCRIPTION	Page. 9
WARNINGS	Page 11
IDENTIFICATION / DIMENSIONS	Page 12
FOOT CONTROL	Page 13
ROTATION	Page 14
LEG-REST	Page 14
ARMREST	Page 14
BACKREST	Page 14
CLEANING AND DISINFECTION	Page 15
ROUTINE MAINTENANCE	Page 16
LEG SUPPORT SLIDING ADJUSTMENT	Page 16
SPARE PARTS LIST	Page 17
CONFORMITY DECLARATION	Page 18

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TECHNICAL SPECIFICATIONS

GB

Power supply	230 VAC single phase/50Hz
Operating voltage	24 volts
Podiatry chair absorption	350W
Allowed mains voltage fluctuation	+/- 10%
Mains fuses	n.2 (T.3,15 A /250V)
Electrical protection rating	Class I -Type B
Liquids protection rating	IPX 0
Separate on-off switch	Capacity 16A /trip threshold 10A
Rated lift capacity	160 Kg.
Leg-rest load capacity	Kg. 25 each
Armrest load capacity	Kg. 35 each
Backrest load capacity	Kg. 55
Operating temperature range	10 / 35 °C
Storage and transport temperature	-10 / 70 °C
Operating humidity range	45 / 90 %
Storage and transport humidity	10 / 95 %
Atmospheric pressure	500 / 1060 mbars
Podiatry chair weight	95 Kg
Podiatry chair dimensions	740 x 800 x 1380(H) mm

Weights are indicative only and may vary according to the exact configuration.

D VERWENDETE SYMBOLS	E SÍMBOLOS	GB KEY TO SYMBOLS	
Wechselstrom	Corriente alterna	Alternating Current	
Schutzerdung	Conexión a tierra	Earth connection	
Gerät Klasse I - Typ B	Aparato de clase 1 Tipo B	Class 1 Type B Equipment	
Hochspannung (230 V)	Alta tensión (230 V)	High Voltage (230 V)	
Eingeschaltet (an das Stromnetz angeschlossen)	Activado (conectado a la red de alimentación)	On (Connected to power supply)	I
Ausgeschaltet (an das Stromnetz nicht angeschlossen)	Desactivado (desconectado de la red de alimentación)	Off (disconnected from power supply)	O
Lesen Sie die Gebrauchsanweisung	Leer el manual del usuario	See the operative instructions	
ACHTUNG, vor Gebrauch das Handbuch lesen	¡ATENCIÓN!, consultar el manual de instrucciones antes de la utilización.	WARNING: see instruction manual before use	
Gemäß der Richtlinie 2007/47/EEC "MEDICAL DEVICES" "Medizinprodukte"	En conformidad con la directiva 2007/47/EEC "MEDICAL DEVICES"	Conforms to EEC Directive 2007/47/EEC "MEDICAL DEVICES"	
Technischer Kundendienst (nur von Fachpersonal ausgeführt)	El Servicio postventa (intervención realizada sólo por personal técnico calificado)	Customer Support Service (task to be done by qualified technical staff only)	S.P.V.
Im Autoklav sterilisierbar	Parte que se puede esterilizar en autoclave	Piece than can be sterilized in autoclave	

Entsorgen gemäß den anzuwendenden Gesetzen. Wenn Sie Fragen oder Bedenken haben, kontaktieren Sie bitte SMD	Eliminar de acuerdo con la ley aplicable. Si usted tiene alguna pregunta o duda por favor póngase en contacto con SMD	Dispose of in accordance with applicable law. If you have any questions or concerns, please contact SMD	
Diese Seite nach oben	Lado superior	Upper side	
Zerbrechlich	Frágil	Fragile	
Vor Nässe schützen	Proteger de la lluvia	Protect from rain	

Die auf dem Gerät erscheinenden Symbole entsprechen der IEC-Norm 601.1 und der ISO-Norm 9687.
 Los símbolos que aparecen en el equipo cumplen con las normas IEC 601.1 e ISO 9687.
 The symbols that appear on the equipment comply with IEC 601.1 and ISO 9687 standards.

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GB GENERAL INSTRUCTIONS

Make sure that the chair is installed on a horizontal plane.
 Keep the base constantly clean to avoid accumulation of organic deposits within the coverings and consequent stinking odours.
 Absolutely avoid pouring of liquids onto the base; in case of accidental pouring, dry as soon as possible.

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⚠ WARNINGS**Purpose of the chair**

Device used for patient support during podiatry interventions. These chairs have been designed and constructed for the use with patients of all ages.

Neither the podiatry unit nor the podiatry chair can be used in presence of flammable anaesthetic gas mixtures.

When treating patients with pace-makers or similar devices the operator must take the necessary precautions: refer to the specific literature on the matter.

Do not exceed the loads indicated in the "technical specifications" section.

Do not remove the labels on any unit/chair devices. Should these deteriorate replace them.

Power supply

Make sure that the unit/chair electrical power supply is powered via an external differential cut-out switch with a 16A current-carrying capacity and a 10mA trip threshold.

Make sure that chair has a separated power supply.

Make sure that the podiatry chair power supply be fitted with an efficient grounding (earth) system and that wiring complies with the standards in force.

Climate

Under extreme conditions (heat, cold, humidity) it is advisable to let a few hours pass between unpacking the equipment and using it for the first time. This precaution provides the time needed to eliminate any condensation that may have formed inside the packaging.

Precautions

The unit needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information contained herein.

The portable devices may affect the operation of the device.

Use only accessories that are specified in this document in order to ensure the maintenance of the EMC performance of the device.

Do not use together with other nearby devices to avoid mutual interference.

Attention: the connection of the electrical power strip leads to the creation of a SYSTEM EM, and this situation may result in a reduced level of security. About the requirements applicable to an EM SYSTEM, the ORGANIZATION RESPONSIBLE must refer to this standard.

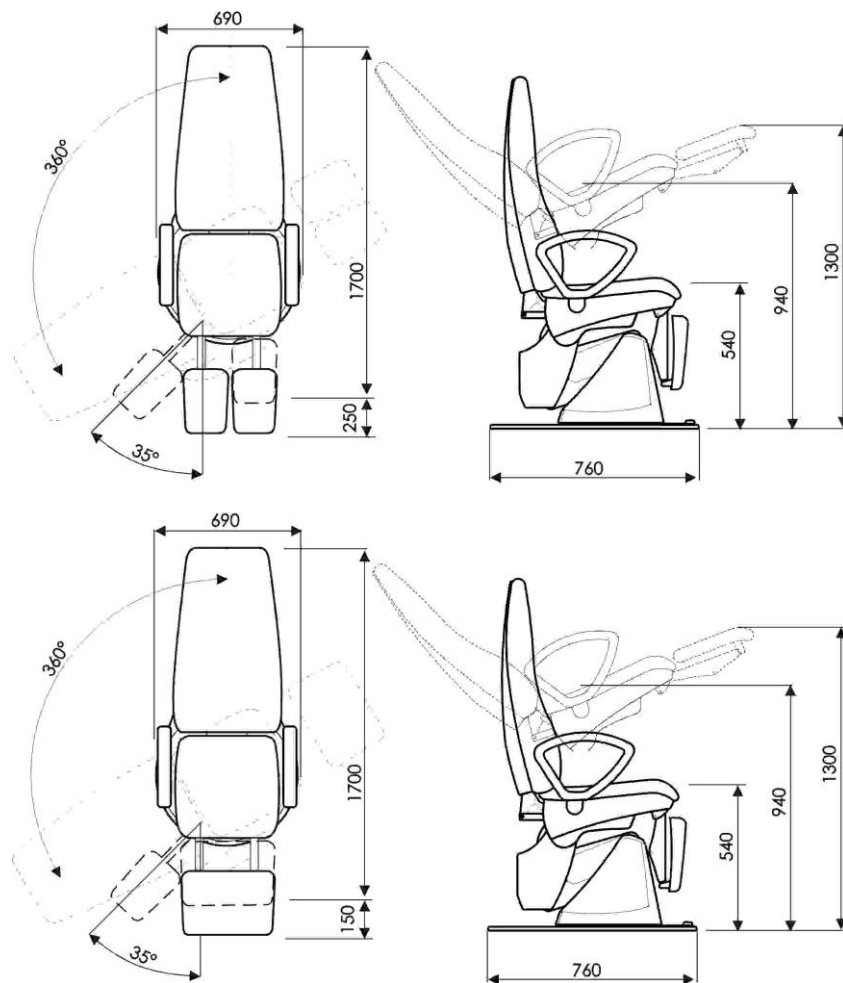
Upon request S.M.D. provides circuit diagrams, spare parts list, descriptions, calibration instructions or other information to assist the technical support personnel during installation or repair.

Should you find any fissures or cracks on the power cord or should it be partly cut off, get SMD qualified service personnel to change it.

S.M.D. cannot be held liable for any damages deriving from failure to observe the warnings and instructions contained in this handbook.

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Are devices of Class I, classification rule 13, annex VIII of the 745/2017 MDR (Medical Device Regulation).
They comply with the essential requirements of the 745/2017 MDR (Medical Device Regulation).
They comply with the RoHS 2011/65/EU Directive (restriction of the use of hazardous substances in electrical and electronic equipment)
They comply with the RED 2014/53/EU Directive (Radio Equipment Directive)

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